

Flash & Spark Network Setting

❖ How to Network Setting

- 1. Connected PC & CR Scanner.
- 2. Click the setting on the Right top in Quantor Software.
- 3. Click the Scanner control in setting.
- 4. Check the check box in Edit parameters. ①
- 5. Change the IP Address & Subnet mask, Gate way. ②
- 6. Click the Save params. ③
- 7. Turn off the CR Scanner & Quantor software.
- 8. Connect the Network cable & remove the USB cable.
- 9. Turn on the CR & Quantor Software.

❖ The CR Scanner and the computer must be on the same network(Router).

